## QUICKTIP

## Faldalarm og advarsler

## Brugerguide

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### **Fall Alert Notifications are not a Substitute for Emergency Services and will not Contact Emergency Services**

Fall Alert notifications are merely a tool that may assist in communicating certain information to one or more third-party contacts the user has identified. My Starkey does not communicate with emergency services or provide emergency assistance in any way and is not a substitute for contacting professional emergency services. The operation of My Starkey's fall-detection features depends on wireless connectivity for both the user and the user's designated contact(s), and the feature will not successfully deliver a message if Bluetooth® or cellular connectivity is lost or interrupted at any point in the communication pathway. Connectivity can be lost under a number of circumstances, such as: a paired mobile device is out of range of the hearing aid(s) or otherwise loses connectivity with the hearing aid(s); the hearing aids or mobile device are not turned on or sufficiently powered; a mobile device is in airplane mode; a mobile device malfunctions; or if bad weather interrupts a mobile device's network connectivity.

### **Fall Alert Feature is a General Wellness Product (Not Regulated as a Medical Device)**

The Fall Alert feature is designed and distributed as a General Wellness product. The Fall Alert feature is not designed or in any way intended to detect, diagnose, treat, cure, or prevent any specific disease or particular, medical condition and is not targeted to any specific or particular population. Rather, the Fall Alert feature is designed only to detect that a user may have fallen and try to send a text message in response to such an event, in support of the user's general health.

Additional information can be found in the operations manual that comes with the hearing aid and the My Starkey End User License Agreement, which is available in My Starkey and must be read and agreed to before using My Starkey.

*Features may vary by country.*

This app may have slight differences depending on your phone.

My Starkey and the Starkey logo are trademarks of Starkey Laboratories, Inc.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Starkey is under license.

Apple, the Apple logo, iPhone, iPod touch, App Store and Siri are trademarks of Apple, Inc., registered in the U.S. and other countries.

©2023 Starkey Laboratories, Inc. All Rights Reserved. 2/23 FLYR4087-00-EN-ST