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---

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# Warnings, Cautions, & Notices

## REQUIRED HEARING AID INFORMATION

The following additional information is provided in compliance with U.S. Food and Drug Administration (FDA) regulations:

**⚠️ WARNING: People younger than 18 should go to a doctor before using this.**

People younger than 18 years old need specialized care, and using this without a medical evaluation may worsen impairment or disability. A hearing aid user who is younger than 18 should have a recent medical evaluation from a doctor, preferably an ear-nose-throat doctor (an ENT). Before using this, a doctor should determine that the use of a hearing aid is appropriate.

**⚠️ WARNING to Hearing Aid Dispensers:**

You should advise a prospective hearing aid user to consult promptly with a doctor, preferably an ear specialist such as an ENT, before dispensing a hearing aid if you determine through inquiry, actual observation, or review of any other available information concerning the prospective user, that the prospective user has any of the following conditions:

- Visible deformity of the ear, either congenital or traumatic
- Fluid, pus, or blood coming out of the ear within the previous 6 months
- Pain or discomfort in the ear
- History of excessive ear wax or suspicion that something is in the ear canal
- Dizziness, either recent or long-standing
- Sudden, quickly worsening, or fluctuating hearing loss within the previous 6 months
- Hearing loss or ringing (tinnitus) only in one ear or a noticeable difference in hearing between ears
- Audiometric air-bone gap equal to or greater than 15 dB at 500 Hz, 1000 Hz, and 2000 Hz

**⚠️ WARNING to Hearing Aid Dispenser, Outputs over 132 dB SPL:**

You should exercise special care in selecting and fitting a hearing aid with a maximum output that exceeds 132 dB SPL because it may impair the remaining hearing of the hearing aid user.

### International Use

Your hearing aids are approved to operate at a radio frequency that is specific to your country or region and might not be approved for use outside your country or region. Be aware that operation during international travel may cause interference to other electronic instruments, or other electronic instruments may cause interference to your hearing aids.

# Warnings, Cautions, & Notices

We are required by regulations to provide the following warnings:

**⚠️ WARNING:** Use of wireless hearing aids directly next to other electronic equipment should be avoided because it could result in improper performance. If such use is necessary, note as to whether your hearing aids and the other equipment are operating normally.

**⚠️ WARNING:** Use of accessories, components or replacement parts other than those provided by the manufacturer of your hearing aids could result in increased electromagnetic emissions and decreased electromagnetic immunity and could result in degradation of performance.

**⚠️ WARNING:** If Portable Radio Frequency communications equipment is used closer than 30 cm (12 inches) from your hearing aid, degradation of the performance of your hearing aid could result. If this occurs, move away from the communications equipment.

**⚠️ WARNING:** The stability of the hearing aid is designed for normal use. Avoid physical impacts to the ear when wearing a hearing aid that may cause the device to break or component of the device to become detached. This may lead to lacerations of the ear canal or perforation of the ear drum. If this occurs, it is strongly recommended to see a physician for evaluation and safe removal.

**⚠️ WARNING:** If the hearing aid drops onto a hard surface, it could cause the hearing aid to break or become damaged. This includes mechanical stress or shock of the device. Ensure the hearing aid is intact before placing it in the ear. If you find the device is damaged, discontinue use and see your hearing professional.

**⚠️ WARNING:** Repeated stress to the removal handle may cause it to break. If the removal handle breaks and you're unable to remove the device carefully, it is recommended to see a physician for safe removal.

**⚠️ WARNING TO HEARING CARE PROFESSIONAL**

A hearing care professional should advise a prospective sound generator (tinnitus masker) user to consult promptly with a licensed physician (preferably an ear specialist) before using a sound generator if the hearing care professional determines through inquiry, actual observation or review or any other available information concerning the prospective user that the prospective user has any of the following conditions:

- Visible congenital or traumatic deformity of the ear
- History of active drainage from the ear within the previous 90 days
- History of sudden or rapidly progressive hearing loss within the previous 90 days
- Acute or chronic dizziness
- Unilateral hearing loss of sudden or recent onset within the previous 90 days

## Warnings, Cautions, & Notices

**⚠ WARNING:** There are some potential concerns associated with the use of any sound generating tinnitus therapy instrument. Among them are the potential for worsening of tinnitus, a possible change in hearing thresholds, and possible skin irritation at the point of contact with the hearing aid.

Multiflex Tinnitus Technology has been designed to minimize these concerns. However, should you experience or notice any of the above conditions or any dizziness, nausea, headaches or heart palpitations, you should immediately discontinue use of the hearing aid and seek a consultation with a medical, audiology or other hearing care professional.

As with any hearing aid, misuse of the tinnitus therapy instrument could present some potentially harmful effects. Care should be taken to prevent the unauthorized use and to keep the hearing aid out of the reach of children and pets.

### FALL ALERT FEATURE

The Fall Alert feature can be used to notify others should you fall or experience a non-fall-related event. This feature can be configured to send an SMS text message to predefined contacts. Fall Alert can be configured to send automated and/or manually-initiated alerts.

**⚠ WARNING:** Auto Alert may not detect 100 percent of falls

**⚠ WARNING:** Decreasing the Auto Alert Sensitivity may prevent some falls from being detected by your Fall Alert system. For example, Auto Alert may not detect a fall if:

- The Sensitivity setting is not appropriate for the user
- The fall is very slow, or you slide down gradually
- You get up and begin walking immediately after a fall

**⚠ WARNING:** Auto Alert may initiate false alerts. To prevent false-alert text messages from being sent to your contact(s), you may cancel the alert from either your smartphone or by pressing the user control on either hearing aid.

**⚠ WARNING:** To reduce Fall Alert communication failures:

- Your hearing aid(s) need to be powered on, paired and connected with your smartphone using Bluetooth® connectivity
- The mobile device must be powered on, with the hearing aid mobile app open (in the foreground or background)
- The mobile device must have a connection to the Internet (via a cellular network or WiFi)

**⚠ CAUTION:** This is not hearing protection.

You should remove this device if you experience overly loud sounds, whether short or long-lasting. If you're in a loud place, you should use the right kind of hearing protection instead of wearing this device. In general, if you would use ear plugs in a loud place, you should remove this device and use ear plugs.

## Warnings, Cautions, & Notices

**⚠ CAUTION:** The sound output should not be uncomfortable or painful.

You should turn down the volume or remove the device if the sound output is uncomfortably loud or painful. If you consistently need to turn the volume down, you may need to further adjust your device.

**⚠ CAUTION:** You might need medical help if a piece gets stuck in your ear.

If any part of your hearing aid, like the eartip, gets stuck in your ear, and you can't easily remove it with your fingers, get medical help as soon as you can. You should not try to use tweezers or cotton swabs because they can push the part farther into your ear, injuring your eardrum or ear canal, possibly seriously.

**NOTE:** What you might expect when you start using a hearing aid.

A hearing aid can benefit many people with hearing loss. However, you should know it will not restore normal hearing, and you may still have some difficulty hearing over noise. Further, a hearing aid will not prevent or improve a medical condition that causes hearing loss.

People who start using hearing aids sometimes need a few weeks to get used to them. Similarly, many people find that training or counseling can help them get more out of their devices.

If you have hearing loss in both ears, you might get more out of using hearing aids in both, especially in situations that make you tired from listening—for example, noisy environments.

**NOTE:** Tell FDA about injuries, malfunctions, or other adverse events.

To report a problem involving your hearing aid, you should submit information to FDA as soon as possible after the problem. FDA calls them "adverse events," and they might include: skin irritation in your ear, injury from the device (like cuts or scratches, or burns from an overheated battery), pieces of the device getting stuck in your ear, suddenly worsening hearing loss from using the devices, etc.

Instructions for reporting are available at <https://www.fda.gov/Safety/MedWatch>, or call 1-800-FDA-1088. You can also download a form to mail to FDA.

**NOTE:** Hearing loss in people younger than 18.

- People younger than 18 should see a doctor first, preferably an ear-nose-throat doctor (an ENT), because they may have different needs than adults
- The doctor will identify and treat medical conditions as appropriate
- The doctor may refer the person to an audiologist for a separate test, a hearing aid evaluation
- The hearing aid evaluation will help the audiologist select and fit the appropriate hearing aid

## Warnings, Cautions, & Notices

A person who is younger than 18 years old with hearing loss should have a medical evaluation by a doctor, preferably an ENT, before buying a hearing aid. The purpose of a medical evaluation is to identify and treat medical conditions that may affect hearing but that a hearing aid won't treat on its own.

Following the medical evaluation and if appropriate, the doctor will provide a written statement that the hearing loss has been medically evaluated and the person is a candidate for a hearing aid. The doctor may refer the person to an audiologist for a hearing aid evaluation, which is different from the medical evaluation and is intended to identify the appropriate hearing aid.

The audiologist will conduct a hearing aid evaluation to assess the person's ability to hear with and without a hearing aid. This will enable the audiologist to select and fit a hearing aid for the person's individual needs. An audiologist can also provide evaluation and rehabilitation since, for people younger than 18, hearing loss may cause problems in language development and educational and social growth.

An audiologist is qualified by training and experience to assist in the evaluation and rehabilitation of hearing loss in people younger than 18.

### **IMPORTANT NOTICE FOR PROSPECTIVE Rx HEARING AID USERS:**

It is good health practice for a person with a hearing loss to have a medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear) before purchasing a prescription hearing aid. Licensed physicians who specialize in diseases of the ear are often referred to as otolaryngologists, otologists or otorhinolaryngologists. The purpose of the medical evaluation is to assure that all medically treatable conditions that may affect hearing are identified and treated before the hearing aid is purchased.

Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for a hearing aid. The physician will refer you to an audiologist or hearing aid dispenser, as appropriate, for a hearing aid evaluation.

The audiologist or hearing aid dispenser will conduct a hearing aid evaluation to assess your ability to hear with and without a hearing aid. The hearing aid evaluation will enable the audiologist or dispenser to select and fit a hearing aid to your individual needs.

If you have reservations about your ability to adapt to amplification, you should inquire about the availability of a trial-rental or purchase option program. Many hearing aid dispensers now offer programs that permit you to wear a hearing aid for a period of time for a nominal fee after which you may decide if you want to purchase the hearing aid.

In some geographies, you must have a medical evaluation before purchasing a prescription hearing aid. Some States allow an adult to waive the medical evaluation.

## Warnings, Cautions, & Notices

A hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Use of a hearing aid is only part of hearing habilitation and may need to be supplemented by auditory training and instruction in lip reading. In most cases infrequent use of a hearing aid does not permit a user to attain full benefit from it.

Some hearing instrument users have reported a buzzing sound in their hearing instrument when they are using mobile phones, indicating that the mobile phone and hearing instrument may not be compatible. It is well-known that mobile phones are potential sources of noise for hearing aids. Your Starkey Hearing Aids have been tested for compliance to two standards that define hearing aid immunity to digital wireless devices and meet the requirements of ANSI C63.19-2019 as well as the criteria for user compatibility as defined by IEC 60118-13:2019.

**⚠ CAUTION: The following are potential physiological side effects from use of hearing aids. Consult a physician if the following occur:**

- Worsening of Hearing Loss or Tinnitus
- Pain or Discomfort from heat (burns), abrasion (cuts and scratches), infection, shock
- Dermal Allergic Reaction (inflammation, irritation, swelling, discharge)
- Excessive production of cerumen (Earwax)

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### WIRELESS NOTICES

mRIC R:  
FCC ID: EOA-24GENMICRO  
IC: 6903A-24GENMICRO

RIC RT:  
FCC ID: EOA-24GENMACRO  
IC: 6903A-24GENMACRO

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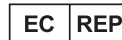
## Regulatory information

For service or repair, please contact your hearing care professional first. If necessary, you can send your hearing aids to:

 **Starkey Laboratories, Inc.**  
6700 Washington Ave. South  
Eden Prairie, MN 55344 USA  
[www.starkey.com](https://www.starkey.com)

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UK Authorised Person:  
Starkey UK  
William F. Austin House  
Pepper Rd, Hazel Grove  
Stockport SK7 5BX, UK  
[www.starkey.co.uk](https://www.starkey.co.uk)

Waste from electronic equipment must be handled according to local regulations

Consult user manual

Keep dry

### Instructions for Disposal of Old Electronics

Starkey Laboratories, Inc. encourages, the EU requires, and your local community laws may require, that your hearing aids be disposed of via your local electronics recycling/disposal process.

The instructions below are provided for the benefit of disposal/recycling personnel. Please include this manual when disposing of your hearing aids.

### FOR DISPOSAL/RECYCLING PERSONNEL ONLY

These products contains a Lithium Ion Polymer battery. For instructions on removing the battery from the hearing aids, please visit <https://web.starkeypro.com/conformity/regulatory-information.html>.

# Regulatory information

Symbol	Symbol Meaning	Applicable Standard	Symbol Number
	Manufacturer	BS EN ISO 15223-1:2021	5.1.1
	Authorized representative in the European Community	BS EN ISO 15223-1:2021	5.1.2
	Date of Manufacture	BS EN ISO 15223-1:2021	5.1.3
	Catalogue Number	BS EN ISO 15223-1:2021	5.1.6
	Serial Number	BS EN ISO 15223-1:2021	5.1.7
	Medical Device	BS EN ISO 15223-1:2021	5.7.7
	Keep Dry	BS EN ISO 15223-1:2021	5.3.4
	Temperature Limit	BS EN ISO 15223-1:2021	5.3.7
	Humidity Limitation	BS EN ISO 15223-1:2021	5.3.8
	Caution	BS EN ISO 15223-1:2021	5.4.4
	General warning sign	EC 60601-1, Reference no. Table D.2, Safety sign 2	ISO 7010-W001
	Refer to instruction manual/booklet	EC 60601-1, Reference no. Table D.2, Safety sign 10	ISO 7010-M002
	Collect Separately	DIRECTIVE 2012/19/EU (WEEE)	Annex IX
	Class II equipment	IEC 60417 Reference no. Table D.1	Symbol 9 (IEC 60417-5172)
	Regulatory Compliance Mark (RCM)	AS/NZS 4417.1:2012	N/A
	Giteki Mark	Japanese Radio Law	N/A
	Direct current	IEC 60601-1 Reference no. Table D.1	IEC 60417-5031
	Recycling Symbol	European Parliament and Council Directive 94/62/EC	Annex I-VII

# Notes

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

Download on the  
**App Store**

GET IT ON  
**Google Play**

---

Use of the Made for Apple badge means that an accessory has been designed to connect specifically to the Apple product(s) identified in the badge, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Apple, the Apple logo, iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license.

Google Play is a trademark of Google LLC.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Starkey is under license. Genesis, Starkey, and StarLink are trademarks of Starkey Laboratories, Inc.